



microcure

**Microcure Medical Technology
Co.,Ltd.**

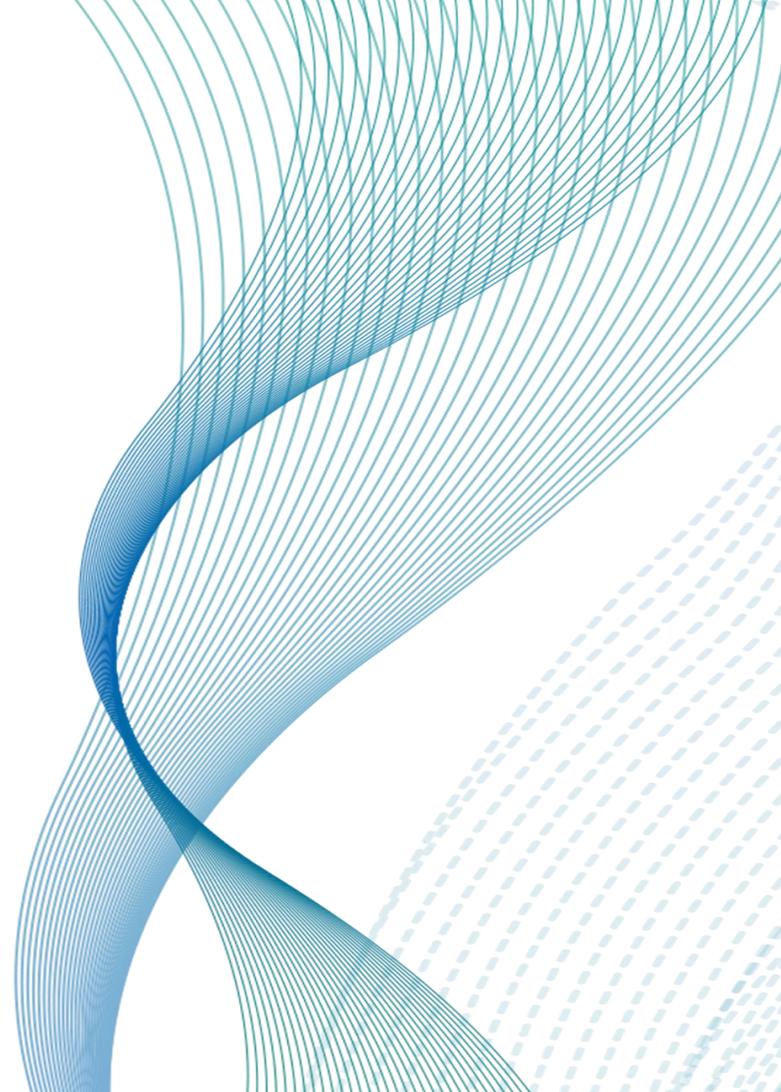

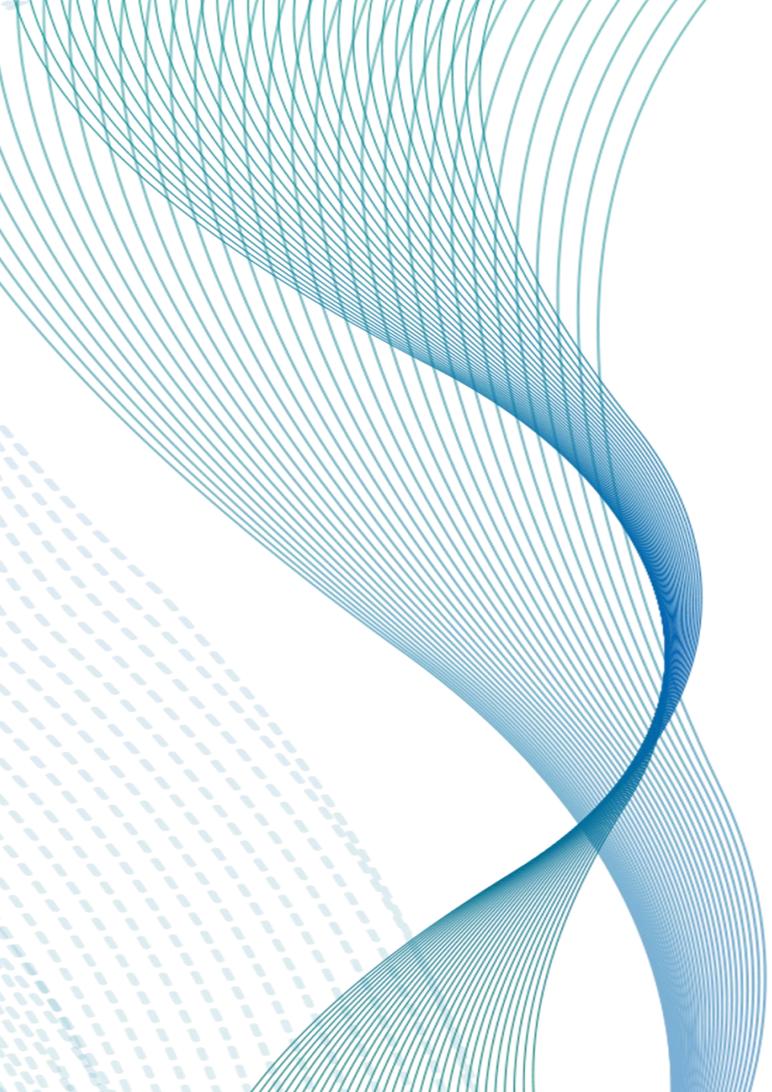

CONTENTS

**1. Company
Introduction**

**2. Clinical
Problem**

**3. Scheme
Introduction**

**4. Using
Method**

**5. Competition
Comparison**

01

**Company
Introduction**

microcure



COMPANY PROFILE

- Established in Jul. 2015
- Locates in Jiangsu Medical Device Park, Suzhou technology Park
- Registered Capital: 3 Million USD
- Received 10 class II MD registration certificates from CFDA in Jan. 2018, CE & ISO13485 & FSC

Company Introduction

“The essence always dwells on smallest granularity, no tiny bit should be ignored.”

This old motto comes from the earliest Chinese history book written about 3000 years ago. It shows an attitude of attention and reasoning of smallest details to receive new knowledge and growth. In this digital era, we are facing and have to embrace the granular society, therefore the technological innovation ought to pay more attention to details, and humanity ought to be more meticulous. MicroCure insists on this idea with firm determination to service our customers and patients with our best efforts.



Company Introduction



microcure

Office area and rest area

Company Introduction



Inspected by CFDA every year



Production area 2400m²
100,000class cleanroom 1200m²



Company Introduction



Physics and
Chemistry
Testing Lab



Servo Universal
Material Tester



Microbiology Lab




Gas
Chromatograph


02 Clinical Problem

microcure

Clinical
Problem

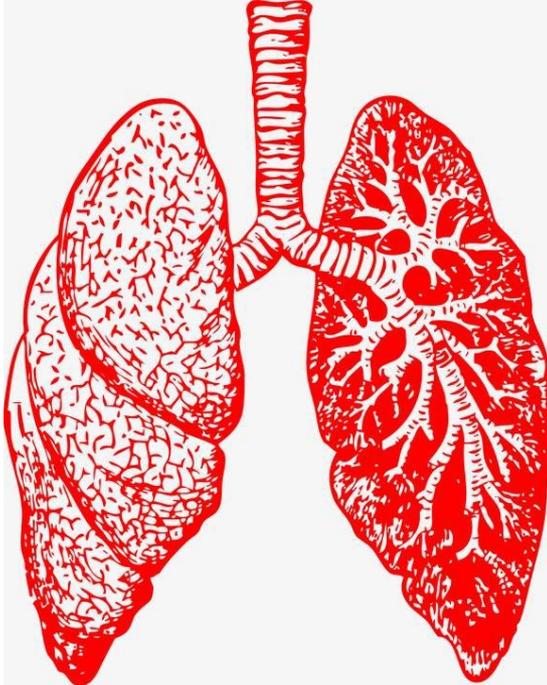

Pulmonary resection

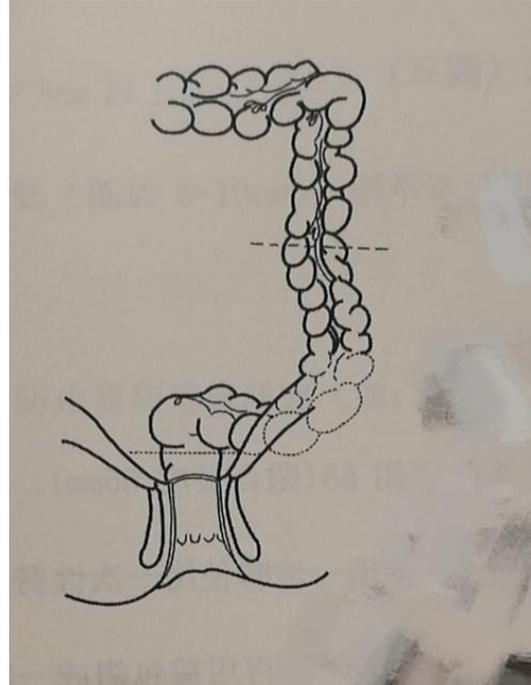

Low rectal resection

03 Scheme Introduction

microcure

Disposable Linear Stapler

Nail anvil

Nail bin assembly

Reset button



Firing handle

Feed handle



Linear stapler

Nail bin: two rows of nails, no knife
microcure

Two rows of left and right titanium nails are provided. The tissue between the two groups of titanium nails can be closed and cut simultaneously.

Disposable
Linear
Stapler



Cutting length: 30/45/60/90
Nail height: L/H

Linear cutting stapler





Cutting length: 30/40
Nail height: L/H

04 Using Method


microcure

Step 1: Remove the safety cover

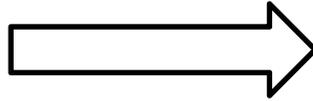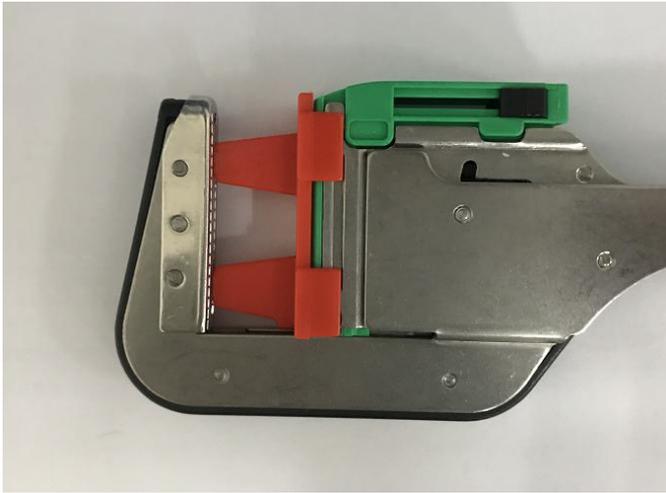

Using Method

Step 2: Place the closed tissue and position it in first gear.

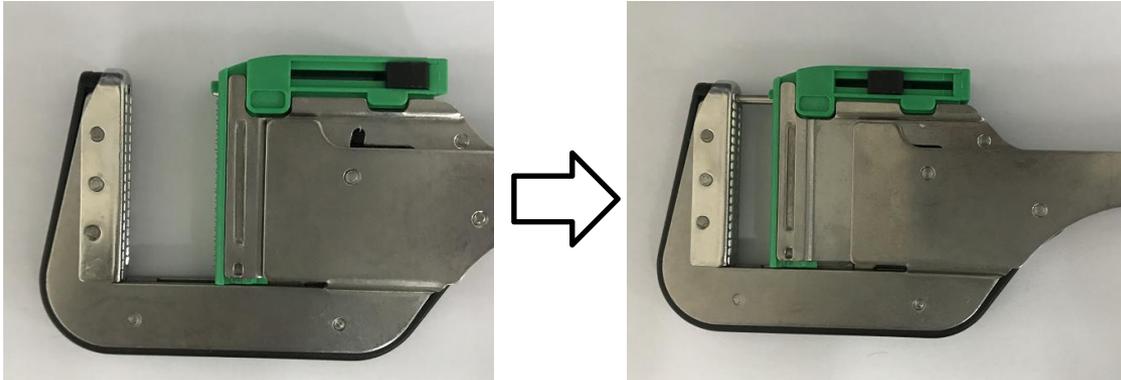

Reset button

Firing handle Feed handle

Using Method

Step 3: 2nd gear, squeezing tissue



Reset button

Feed handle


Step 4: Firing



microcure



Step 5: Remove the stapler

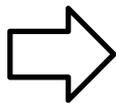

In perfection we believe

microcure

<http://www.microcuremed.com.cn/>

Email: export@microcure.com.cn

